

St. John's Medical College Hospital, Bangalore 560034
Face Shield In-house Production

Face Shield

To make additional face protection for high risk of aerosol procedures

Note: Used along with full body suit, N95 mask and goggles

Requirement:

1. Disposable hole towel
2. Transparent OHP sheet
3. Staplers

Procedure:


1. Hole towel



2. Fold upper end of hole towel till hole



3. Place OHP sheet in hole and staple it

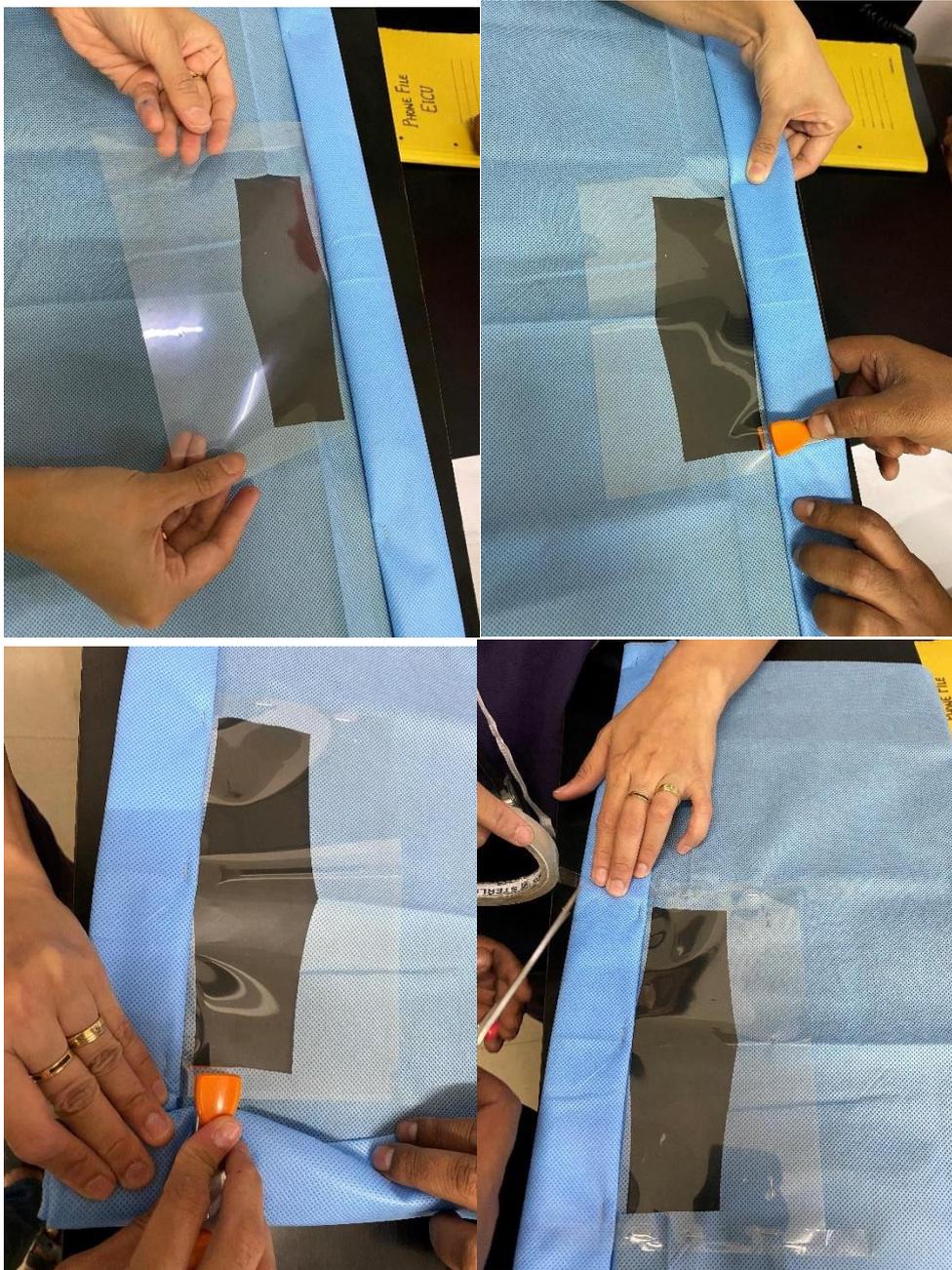

Note: the OHP can be firmly secured with tapes or glue or staples to avoid gaps between the OHP sheet and the hole towel

4. Use as demonstrated, can be secured by stapler



Note: one can replace the hole towel with plastic or other fluid impervious material.

Thank you, keep safe.